



Flowable Adhesive Calcium

## Base / Liner

### Directions for Use:

#### Intended Use:

Ca-Lok is a resin-based, flowable composite recommended as a lining material beneath composite resins or as a preventive resin restoration (PRR) material. Ca-Lok is fluoride releasing and contains calcium. Because Ca-Lok is highly radiopaque this allows for easy radio-graphic observation compared to enamel, dentin, and possible secondary caries.

Ca-Lok is provided in a base white tooth shade. Ca-Lok contains Triclosan. No Clinical Testing has been performed to demonstrate antimicrobial effectiveness.

#### Instructions for Use:

1. Prepare cavity preparation as required to remove decay and according to established procedures.
2. Remove cap of Ca-Lok syringe and place a fresh applicator tip on the syringe.
3. Gently advance the syringe barrel to extrude the needed amount of Ca-Lok to the desired site.
4. Smooth out Ca-Lok over the desired area if needed and cure the material using a dental curing light with a power of at least 300 mW/cm(2) until material is cured. Typically 20 seconds or less depending upon the power of the curing light.
5. Examine the cured material to insure that adequate coverage has been achieved.
6. Replace the syringe cap on the syringe, safely dispose of the applicator tip and proceed with cavity filling using the appropriate restorative material.

#### Storage Conditions, Precautions and Disposal:

Those individuals with a known sensitivity to acrylate resins, or Triclosan, should avoid using this product. Dental personnel should wear appropriate personal protective equipment (PPE). Dispose of old, outdated material properly. Avoid skin contact and clean-up with paper towels as needed.

*For Use only by a Licensed Dentist. Rx Use Only.*



277 NEW YORK AVENUE, JERSEY CITY, NJ 07307

TELE: 800.828.2634 / 201.798.5353 FAX: 201.659.7186

EMAIL: SALES@TAUBDENTAL.COM

WEBSITE: [HTTP://WWW.TAUBDENTAL.COM](http://www.taubdental.com)