

DESCRIPTION AND INTENDED USE:

Kidz Seal - America Pit & Fissure Sealant is used to seal the pits and fissures of teeth. Kidz Seal - America is comprised of hydrophilic resins and a balance of hydrophilic and hydrophobic fillers. Together these provide optimum properties for sealing and excellent tooth wetting. Kidz Seal - America has an extremely high compressive strength. This strength combined with excellent sealing and great esthetics makes it the perfect pit & fissure sealant.

DIRECTIONS FOR USE:

1. **Clean:** Clean the surface to be sealed with an oil-free prophylaxis paste. Rinse and dry thoroughly.
2. **Isolate:** It is important that tooth surfaces to be restored or sealed be kept free of contamination throughout the procedure.
3. **Etch:** Apply etchant for 15 seconds to the tooth surface to be sealed. Rinse with water and lightly dry.
4. **Apply Sealant:** Place applicator tip on syringe and start flow of Sealant. Use a new applicator tip for each patient. Apply Sealant to surface, lesions, cervical erosion, or pits and fissures, as indicated.
5. **Light Cure:** Light cure for 20 seconds.
6. **Inspect the sealed surfaces:** If tooth has been contaminated, re-etch for 10 seconds, wash, lightly dry and place additional material. Cure for 20 seconds.
7. **Check occlusion:** Adjust as required.

WARNING:

- For professional use by dental personnel only.
- Sealant and Etchant may cause irritation to eyes and skin on contact. Take all necessary precautions to avoid contact with eyes and soft tissue. If materials contact eyes, rinse with water. If materials contact skin, wash skin with soap and water. Seek medical attention if irritation persists.
- Wear protective eyewear and gloves. Wash hands after use. Emergency eye wash fountain should be available.

STORAGE

Store tightly capped in original container at cool room temperature. Avoid direct sunlight and extremes of temperature. Re-cap immediately after use. Designed to be used at room temperature of 21-24° C or 70-75° F.